

Alcohol Surface Disinfectant Wipes

60sec

Lemon

x200

! Always Read Product Label and Material Safety Data Sheet before use. For Professional use only.

- Large size surface disinfectant wipes for disinfecting surfaces and non-invasive medical devices.
- Alcohol-based disinfectant (63% ethanol)
- Wipes have a lemon aroma and supplied in packs of 200. Each wipe measures 200 x 200mm.
- Bactericidal, Yeasticidal, Fungicidal and Tuberculocidal. Also effective against viruses such as Coronavirus, Adenovirus, Norovirus, HIV and Hep C with a contact time of only 60 seconds.

g9chemicals.com



Your only solution

Alcohol Surface Disinfectant Wipes

60sec

Lemon

x200

ⓘ Always Read Product Label and Material Safety Data Sheet before use. For Professional use only.

Contact Times

Bactericidal – 60 sec
Yeasticidal – 60 sec
Enveloped Viruses – 60 sec
Non enveloped Viruses 60 sec
Mycobactericidal – 60 sec
Tuberculocidal 60 sec
Fungicidal – 5 mins

Disinfectant Test Portfolio

The following test standards have been achieved: EN16615, EN14885, EN14476, EN14348, EN13727, EN13624

Bactericidal:

Pseudomonas aeruginosa,
Enterococcus hirae,
Staphylococcus aureus, MRSA,
VRE, E-coli

Yeasticidal:

Candida albicans

Enveloped Viruses:

Hepatitis B virus (HBV),
Hepatitis C virus (HCV), Human
Immunodeficiency virus (HIV),
Human T Cell Leukemia Virus
(HTLV), Influenza virus (which
includes human, avian and porcine
influenza viruses), Hepatitis
delta virus (HDV), Coronavirus,
Filoviridae (which includes Ebola
virus), Rubella virus, Measles virus,
Herpesviridae, Flavivirus, Rabies
virus, Poxviridae

Non enveloped Viruses:

Adenovirus, Norovirus

Fungicidal:

Aspergillus brasiliensis

Mycobactericidal:

Mycobacterium avium

Tuberculocidal:

Mycobacterium terrae

Directions For Use:

Using a new wipe for each application, wipe objects to be disinfected so that they are thoroughly moistened and let the disinfectant take action over a minimum of 60 seconds (to achieve full efficacy, leave for five minutes). Allow surface to dry or wipe with clean tissue.

g9chemicals.com



G9 Chemicals Ltd, 24 High St,
Charing, Ashford, Kent,
TN27 0HX, UK

+44 (0)1580 895 141
info@g9chemicals.co.uk
g9chemicals.com