

INSTRUCTIONS FOR USE

Alcohol-free Surface Wipes

For the surface disinfection of non-invasive medical devices.
Contains detergent.

! Always Read Product label and Material Safety Data Sheet before use. For Professional use only.

! Always wear required PPE when handling chemicals. Appropriate goggles, face shield, mask, gloves and apron.



User Tub

1

Remove cap of dispenser tub.



2

Cut open air tight plastic pouch across the top of the pouch.



3

Feed the centre wipe through the hole in the cap and replace.



4

Using a new wipe for each application wipe objects to be disinfected so that they are completely moistened.



5

Let the disinfectant take action for a minimum of 60 seconds (to achieve full efficacy leave for 5 minutes), allowing surface moisture to evaporate.



Efficacy

Bactericidal (30 s). MRSA, VRE (60 s). Legionella pneumophila (5 min). Yeastcidal (30 s). Fungicidal (30 s). Clostridium difficile (5 min). Mycobacterium avium (2 min). Virucidal activity against enveloped viruses (2 min): Vaccinia virus, HBV, HCV, Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles. Tested to standards: EN 14885, EN 14476, EN 14348, EN 13727, EN 13704, EN 13624.

Composition

Aqua, Benzyl-C12-18-Alkyldimethyl Chlorides, C12-14-Alkyl[(Ethylphenyl) Methyl]Dimethyl Chlorides,

Didecyltrimethylammonium Chloride, 2-Propanol (not an active, <1%), Surfactant, Dipropylene glycol monomethyl ether, Polyhexamethylene biguanide hydrochloride, Cocamidopropyl Betaine, Tetrasodium EDTA, Antifoam, Preservative, Aroma.

WARNING

P101: If medical advice is needed, have product container or label at hand. P102: Keep out of reach of children. P301+P312: IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.

Continue rinsing. The wearing of protective gloves is recommended.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Intended Users

The products are used within the surgery by trained professionals as part of their general disinfecting routine. Patients are usually not present during use, but all patients that are seen in the surgery contribute to the need for the products.

UFI: 8300-U0KQ-700N-8H2K



Your only solution



Tub widely recycled