

## INSTRUCTIONS FOR USE

# Alcohol Surface Wipes

For the surface disinfection of non-invasive medical devices.

## User Tub

# 1

Remove cap of dispenser tub.




# 2

Cut open air tight plastic pouch across the top of the pouch.



# 3

Feed the centre wipe through the hole in the cap and replace.

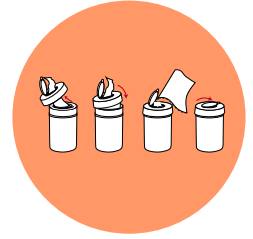


# 4

Using a new wipe for each application wipe objects to be disinfected so that they are completely moistened.



# 5

Let the disinfectant take action for a minimum of 60 seconds (to achieve full efficacy leave for 5 minutes), allowing surface moisture to evaporate.

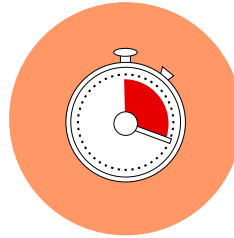

⚠ Always Read Product label and Material Safety Data Sheet before use. For Professional use only.

⚠ Always wear required PPE when handling chemicals. Appropriate goggles, face shield, mask, gloves and apron.



**Efficacy** Bactericidal (60 secs), including MRSA, VRE. Yeastcidal (60 secs). Mycobactericidal (60 secs). Tuberculocidal (60 secs). Fungicidal (5 mins). Viruses to include (60 secs): Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles. Tested to Standards: EN 14885, EN 14476, EN 14348, EN 13727, EN 13624.

**Composition** Ethanol, Aqua, Parfum, Linalool, Limonene. Active: Ethanol 63% w/w (630g/kg).

**DANGER** H228: Flammable solid. H319: Causes serious eye irritation. P280:

Wear protective gloves/protective clothing/eye protection/face protection. P303+P361+P353: IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P370+P378: In case of fire: Use alcohol resistant foam or carbon dioxide to extinguish. Storage. P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. Contraindications. Incompatible materials: Copper, aluminium. Do not use on surfaces sensitive to alcohol. Disposal procedures. Do not discharge liquid

into drains or rivers. Do not flush. Non maceratable.

**Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.**


### Intended Users

The products are used within the surgery by trained professionals as part of their general disinfecting routine. Patients are usually not present during use, but all patients that are seen in the surgery contribute to the need for the products.

Lemon aroma  
UFI: EE00-V0C9-F00M-WVDU  
Natural aroma  
UFI: NH00-D01P-S004-K6YW



Your only solution

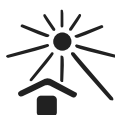

Tub widely recycled

Available at [g9chemicals.com](http://g9chemicals.com)